

INSTRUCTION MANUAL

HUMERUS INTRAOSSEOUS TRAINER

No. 205



Nasco
HEALTHCARE

About the Simulator



Thank you for choosing Nasco Healthcare for your training needs.

No. 205 Humerus Intraosseous (IO) Trainer.

Humerus IO Trainer with bone installed and injection pad removed. The torso can be used in a standing position, on its side, or on its back.

No. 206 Humerus Injection Pad/Bone 3 Pack.



Cover - Front

Cover - Back

Replacement
Bone



The bone can be easily replaced after several uses. Once the bone is replaced return the cover over the bone and the unit is ready for training.

Storing Nasco Healthcare's Training Aids

Nasco Healthcare Manikins should be stored within a temperature range of 60 to 80 degrees Fahrenheit.

Manikins should be stored in a cool, dry environment, lying flat in their storage containers. If manikins are stored in temperatures over 80 degrees Fahrenheit, the manikin material may become pliable and any pressure applied such as weight of the manikin on the head, and any pressure applied may cause distortion, including the weight of the head. Re-warming and allowing cooling without pressure may allow most manikins to revert to their original shape.

Storage below 60 degrees Fahrenheit is not recommended because the manikin material may become brittle and susceptible to cracking, if rough handling is used. Allow the manikin to become room temperature before utilizing.

Warranty

Nasco Healthcare warrants this product to be free from any defect in materials and/or workmanship for a period of three years from the date of purchase, as evidenced by the date of invoice when the product was shipped to the end user. This warranty expressly does not cover abuse, accidental or purposeful damage, or any form of modification to the product.

Nasco Healthcare reserves the right to either repair or replace affected parts, or the entire unit, at their sole discretion, after investigating and reviewing the actual product and the damage. In most instances, a digital photo of the product in question, showing the damage, will help qualify a product for return to the factory. At no time will any product be accepted at the plant without proper return authorization issued by Nasco Healthcare.

Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due. Any product considered for warranty work must be identified by serial number and invoice number from the agency through whom the product was purchased. Without this information the product will not receive a return authorization number as required.

Do not allow print material to come in contact with the skin of your product. The ink will embed in the porous surface and be impossible to remove. Ink stains are not covered under warranty.

Return Policy

Should it be necessary to return an item for credit, or for any other reason, contact our Customer Service Department to obtain an RGA Number. Please refer to your invoice number when phoning in your request for returning merchandise.

Should you have any questions, or wish further information on any product we manufacture, call or write our Customer Service Department.



Nasco Healthcare
16 Simulaid Drive
Saugerties, NY 12477
1-833-NASCOHC (627-2642)
info@nascohealthcare.com
www.nascohealthcare.com